

LOUPES

Instructions For Use



admetec

INTRODUCTION

Thank you for choosing  **admetec**

Your loupes have been designed and custom-made especially for you, using cutting-edge technologies to ensure that your work needs are met with the best solution.

Before attempting to work with your loupes, we recommend that you take the time to carefully read these user instructions.

Loupes are magnifying glasses used by qualified professionals in the dental and healthcare industry to aid them in magnifying the operating area.

Although your new loupes have been fully tested and adjusted by our factory technicians, they may need to be initially adjusted to your facial features.

Please keep in mind;

It is normal to experience initial disorientation when first using magnification. This can also happen when increasing from lower magnification to high magnification power.

A three-week acclimation period is highly recommended for your eyes to adjust and your hand-eye coordination to sync in.

If you experience dizziness, we strongly suggest wearing your loupes for no more than one hour at a time.

INSTRUCTIONS FOR USE

Warnings:

1. Your loupes must be used exclusively for the intended purpose described.
2. Do not use your loupes if any component of it is damaged or detached.

Precautions:

1. Do not dismantle or modify your loupes.
2. Do not spray directly onto the oculars.
3. Do not run under water.
4. Do not submerge in any liquid or ultrasonic cleaner.
5. Do not use a cleaner with greater alcohol percentage than 80%.
6. Do not use machinery sterilization methods to clean your loupes.




Additional information can be found on our website:
www.admetec.com/support

INSTRUCTIONS FOR USE



Adjust Your Loupes:

The frame should sit on the nose bridge level and closer to the eyes to reach the optimal view. Your loupes have been adjusted to fit your exact PD, therefore if any additional adjustments are needed, please use the following instructions:

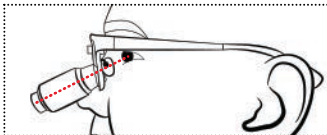



For Too High Telescopes:

Adjust the nose pads width by pressing the nose pads outward. This action will lower the height of the frame and the oculars position.

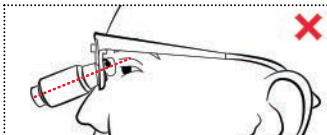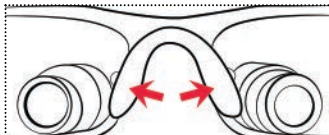

For Too Low Telescopes:

Adjust the nose pad's width by pressing them inward. This action will raise the height of the frame and the oculars position.

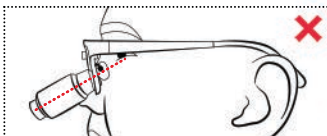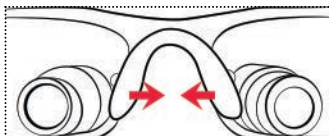

INSTRUCTIONS FOR USE

Headstrap/lanyard Adjustment & Installation:

It is highly recommended to use the included headstrap for your loupes to distribute weight around the head and allow long-time comfort.

1. Insert the headstrap point to the temple tips. [Fig. 05]
2. Loosen the headstrap with the push button and place the frame on your head. [Fig. 06]
3. Tighten the headstrap gently. (Your loupes should fit comfortably on your nose or head.) [Fig. 07]
4. To remove your loupes, simply loosen the headstrap and pull them over your head.

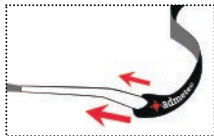


Fig. 05

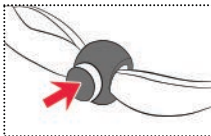


Fig. 06



Fig. 07

Temples Adjustment:

The temple tips on most of our frames are easily adjustable. For more secure and comfortable fit, you can bend the temple tips. [Fig. 08]

1. Align the temple tips and place both thumbs together. [Fig. 09]
2. Slowly and steady bend the temple tips downward until you reach a comfortable fit.
 - Bending temple tips are available on all our frames besides Bolle frame.
3. We recommend visiting your local optometrist for additional adjustments.



Fig. 08



Fig. 09

INSTRUCTIONS FOR USE

Cleaning Instructions:

Your loupes are manufactured from top-quality precision components and will provide you with a long use life with proper care. The included microfiber cloth can be used for cleaning both lenses and loupes.

To avoid scratching the ocular lenses your loupes must be properly cleaned and protected from rough use and improper storage.

1. Spray the cleaning solution directly on the microfiber cloth included.
2. Gently wipe the external surface using the microfiber cloth.
3. While wiping your loupes - do not apply too much force on the lenses.
4. Make sure not to poll any of the components.
5. After finishing, let the microfiber cloth self-dry.
6. Make sure your microfiber cloth is clean and dry before using it.

Prescription Change Service:

In case of eye prescription changes, you can send your loupes to our factory lab to install your new prescription.

When coordinating the return of your loupes, please discuss your schedule with your local distributor to coordinate the return shipment of your loupes. When sending – Make sure you package your loupes and prescription carefully.

PLEASE, DO NOT SHIP YOUR LOUPES BEFORE CALLING TO MAKE ARRANGEMENTS WITH YOUR LOCAL DISTRIBUTOR.

Additional information can be found on our website:
www.admetec.com/support

ADMETEC WARRANTY

Warranty Coverage:

Admetec's warranty obligations for the Magnification Loupes are limited to the terms set forth below:

The products from Admetec come with a warranty. The warranty terms of the Magnification Loupes are stated as below:

The warranty covers defects in material and workmanship for a period of two years, starting from the date of the original purchase. This shall hereinafter be referred to as the Warranty period.

During the Warranty period, if any defect are noticed:

1. Admetec will be responsible to repair the loupes at no extra charge. The repair procedure might use either new or refurbished part/s, based on the availability of the material.
2. Admetec will also be responsible to replace the product with a new one or with another product that has been manufactured from a new or serviceable used part. The replaced part should have equal functional benefits as the original product.

However, if any defect arise after the first ninety days of the Warranty period, shipping and handling charge will be applied to the buyer for repair or exchange of products undertaken by Admetec.

When a product or part is exchanged, any replacement item becomes the buyer's property, and the replaced item becomes Admetec's property.

Warranty Coverage:

This warranty does not apply to the following:

1. Damage caused by accident, abuse, misuse, misapplication, or non-Admetec products.
2. Damage caused by service performed by anyone other than Admetec.
3. Product or part that has been modified without the written permission of Admetec.
4. If any Admetec warranty label has been removed or defaced.

Customer Service
Please call your local distributor.



Admetec Solutions LTD.
Yokhanan HaSandlar St 18
Haifa, Israel



Obelis Group
Bd Général Wahis, 53,
Brussels, B-1030 Belgium



Custom Made