

Cleaning Your Loupes

Do's and Don'ts

Don'ts



Don't rinse or submerge loupes in any liquid or ultrasonic cleaner.

ergoV



Warning:

Ergo V is not waterproof.
Ergo V must be completely dry before switching magnifications.



Avoid sterilizing in autoclaves or chemiclave machines.



Don't use products containing:

- Glutaraldehydes or Iodophors. (Lysol)
- Ammonia
- Bleach
- Acids



Avoid cleaning or introducing moisture into any electronic outlets or exposed metal components connected to your loupes.

Scan for Video



Do's ✓

Be gentle when cleaning the loupes lenses to avoid scratches.

To Clean Your Loupes:



Use Admetec's Official
Lens Cleaning Wipes
Contains 70% Isopropyl Alcohol.

If not available:



Use a soft cloth with alcohol up to 70%.
Or use mild soap and water.



Squeeze out any excess moisture from
the wipe or cloth.



Dry and remove any residue with a soft
cloth or air dry.

admetec
We've got your back

Scan for Video



IM4000008 rev.C