



Loupes

Instructions for Use

IM4000004 Rev.E

Introduction

Thank you for choosing **admetec**

Your loupes have been designed and custom-made especially for you, using cutting-edge technology to ensure that your work needs are met with the best solution.

Before attempting to work with your loupes, we recommend that you take the time to carefully read the instructions in this user manual.

Although your new loupes have been customized and fully tested by our factory technicians, before you begin using them, they may need to be initially adjusted to sit well on your face.

Please keep in mind that it is normal to experience initial disorientation when first using magnification. This can also happen when increasing from a lower to a higher magnification or transitioning from standard loupes to ergonomic loupes.

A three-week acclimation period is completely normal. It is highly recommended to allow time for your eyes to adjust and your hand-eye coordination to synchronize.

If you experience dizziness, we strongly suggest wearing your loupes for no more than one hour at a time.

Warnings

1. Your loupes must be used exclusively for the intended purpose.
2. Do not use your loupes if any component is damaged or has detached.

Precautions:

1. Do not disassemble or modify your loupes.
2. Do not spray anything directly onto the oculars.
3. Do not place loupes under running water.
4. Do not submerge in any liquid or ultrasonic cleaner.
5. Do not use a cleaner with a greater alcohol percentage than 70%.
6. Do not use machinery sterilization methods to clean your loupes.

Prescription change

In case of a change in your prescription, it is possible to send your loupes to our factory for refurbishment.


Please arrange this via your local distributor.

PLEASE DO NOT SHIP YOUR LOUPES BEFORE MAKING ARRANGEMENTS WITH YOUR LOCAL DISTRIBUTOR.

Additional information can be found on our website:
www.admetec.com



Adjust your loupes:

For optimal positioning and view, the frame should sit on the bridge of your nose and closer to your eyes. Your loupes have been crafted to fit your exact pupillary distance therefore, if any additional adjustments are needed, please use the following instructions:

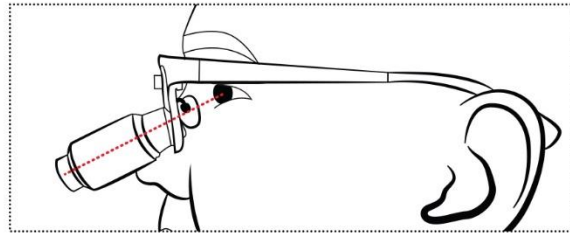



For oculars that sit too high:

Press the nose pads outward. This will increase the space for your nose, thereby lowering the height of the frame and the oculars' positioning.

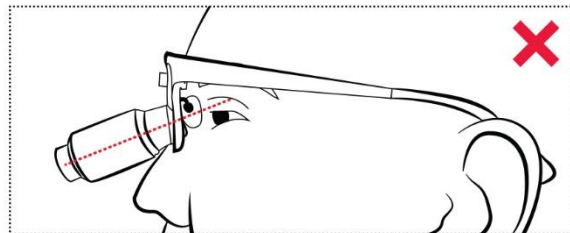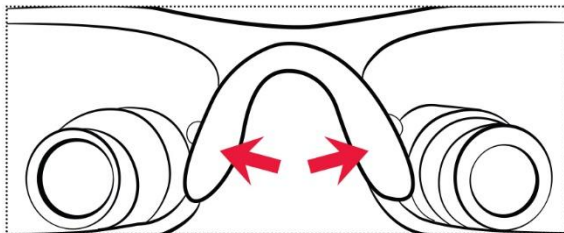

For telescopes that sit too low:

Press the nose pads inward. This will raise the height of the frame and the oculars' positioning.

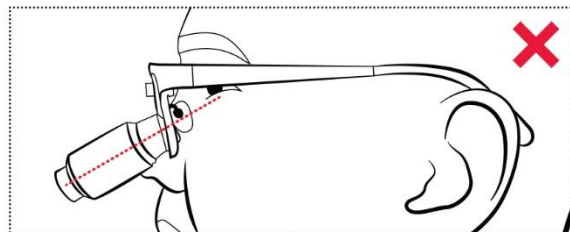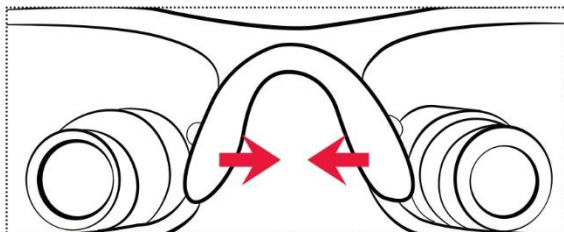

Lanyard adjustment & installation:

It is highly recommended you use the included lanyard for your loupes to evenly distribute their weight and increase your comfort throughout the workday. [Fig. 05]

1. Insert the lanyard tips into the temple-tips holes and snap the buttons shut. [Fig. 06]
2. Adjust the lanyard to your head (loosen or tighten) by pushing and sliding the button to the desired size. [Fig. 07]

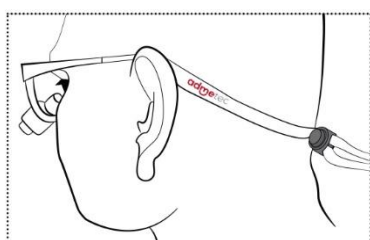


Fig. 05



Fig. 06

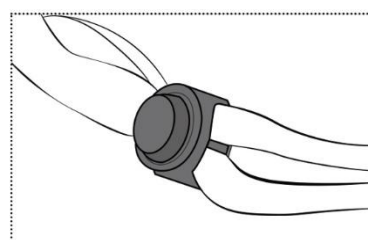


Fig. 07

Temple adjustments:

1. The temple tips on our frames are easy to adjust. For a more secure and comfortable fit, you can bend the temple tips. [Fig. 08]
2. Align the temple tips and place both thumbs together. [Fig. 09]
3. Slowly and steadily bend the temple tips downward until you reach a comfortable fit.
4. We recommend visiting your local optometrist for any additional adjustments.



Fig. 08

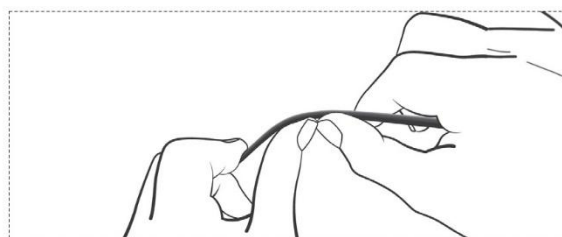


Fig. 09

Adjusting Magnification on Ergo V Oculars

To change magnification, locate the blue rotating wheel near the lenses. The current magnification is marked on the wheel.

While wearing the loupes:

- Turn the wheel clockwise to increase magnification. [Fig.10 and Fig.11]
- Turn counterclockwise to decrease it.

Rotate gently—no force is needed.

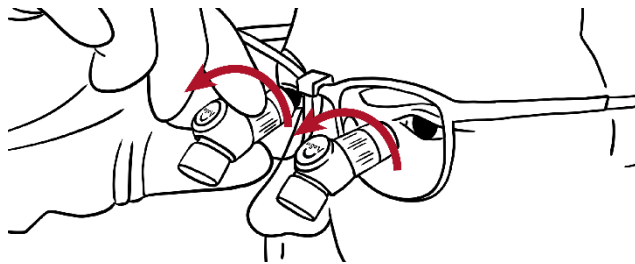


Fig.10

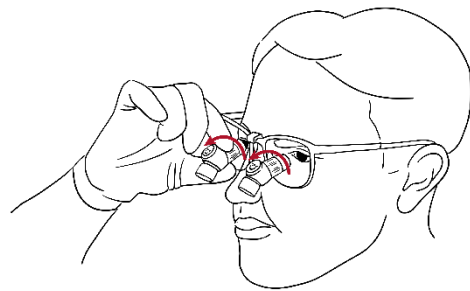


Fig.11

When setting up the loupes for the first time, set the Ergo-V to the highest magnification and adjust your posture to achieve comfortable and clear focus.

Cleaning Instructions

Warning! Improper cleaning methods can damage your loupes. Please follow these instructions carefully to ensure proper cleaning.

Don't



Don't rinse or submerge loupes in any liquid or ultrasonic cleaner.



Avoid sterilizing in autoclaves or chemicalave machines.



Avoid cleaning or introducing moisture into any electronic outlets or exposed metal components connected to your loupes.



Don't use products containing:

- Glutaraldehydes or iodophors. (Lysol)
- Ammonia
- Bleach
- Acids

ergoV



Do not switch magnifications while cleaning Ergo-V or Ergo V-Pro loupes, or until the loupes are completely dry.

Note: Ergo-V is not waterproof—moisture may enter during magnification movement and cause fogging or damage.

Do's



Be gentle when cleaning the loupes lenses to avoid scratches.



To clean your loupes:

Use Admetecs Official
Lens Cleaning Wipes

Contains 70% Isopropyl Alcohol.

If not available:



Use a soft cloth with alcohol up to 70%.
Or use mild soap and water.



Squeeze out any excess moisture from the wipe or cloth.



Dry and remove any residue with a soft cloth or air dry.

WARRANTY

Warranty coverage:

Admetec's warranty obligations for the Magnification Loupes are limited to the terms set forth below:

The warranty covers defects in material and workmanship for a period of two (2) years, starting from the date of the original purchase. This shall, hereinafter, be referred to as the Warranty Period.

During the Warranty Period, if any defect is noticed:

1. Admetec will be responsible for repairing the loupes at no extra charge. The repair procedure might use either new or refurbished part(s), based on the availability of the material.
2. Should it not be possible to make the repair, Admetec will be responsible for replacing the product with a new one or with another product that has been manufactured from a new or serviceable used part. The replaced part will have equal functional benefits as the original product.








Should any defect arise after the first ninety (90) days of the Warranty Period, a shipping and handling charge will be applied to the buyer for repair or exchange of products.

When a product or part is exchanged, any replacement item will become the buyer's property, and the replaced item will become Admetec's property.

Warranty Coverage:

This warranty does not apply to the following:

1. Damage caused by accident, abuse, misuse, misapplication, or by any non-Admetec products.
2. Damage caused by service performed by anyone other than Admetec.
3. A product or part that has been modified without the written permission of Admetec.
4. If any Admetec warranty label has been removed or defaced.

	CE Mark indicating that the device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices
	Admetec Solutions Ltd. Yokhanan HaSandlar St. 18 Haifa, 3296218, Israel
	Obelis Group Bd Général Wahis, 53, Brussels, B-1030 Belgium
	Medical Device
	Caution! Consult accompanying documents
	Batch number
	Catalog number

Please note that the images depicted in this user manual are for illustrative purposes only and may vary between different products versions.

Do you need customer service?

Please contact your local distributor.



Admetec Solutions LTD.
Yokhanan HaSandlar St 18
Haifa, Israel.



Obelis S.A
Bd Général Wahis, 53,
Brussels, B-1030 Belgium.

IM4000004 Rev.E